

Breathbag

Breathbag is a sampling tool for breath tests to be used with Kibion® Dynamic Base and Kibion Dynamic Pro devices. Breathbag is available as single or double variants.

Breathbag is for single use only and should be used together with Mouthpiece.



MANUFACTURER

Kibion GmbH
 Haferwende 31
 28357 Bremen
 Germany
Contact
 +49 (0) 421 27 86 5-0
Support.kibion@mayoly.com
www.kibion.com

Specifications

Product	Single Breathbag	Double Breathbag
Article.No	8004	8005
Package size	50 pcs	50 pcs
Sample volume	1.3 L	2 x 0.1 L
Size	300 x150 mm	200 x 170 mm
Storage and transport	Room temp./Ambient temp., protected from UV-light	
Shelf life	2 years from manufacturing, expiry date is printed on the product	
CE-marking	In vitro diagnostic medical device, CE-marked acc. to 98/79 EC	

Intended use

Breathbag is intended to be used for sampling and collection of breath test samples for in vitro diagnostic purposes of CO₂ analysis. The patient exhales through a Mouthpiece into the Breathbag under supervision of trained professional medical staff in a hospital or medical clinical.

Related products

Product	Article No.
Mouthpiece	8007
Kibion Dynamic Base	8031
Kibion Dynamic Pro	8032

Breathbag

Principle of operation

- Connect a Mouthpiece to the Breathbag tube prior sampling.



- Exhale into the Breathbag through the Mouthpiece.
- Remove the Mouthpiece and close the tube with the blue stopper.
- Remove the blue stopper and connect the Breathbag to Kibion® Dynamic Base or Kibion Dynamic Pro unit for analysis.

Diabact® Urea Breath Test sampling



Exhale into the 0-sample Breathbag



Take a Diabact tablet with a glass of water.
Wait for 10 minutes



Exhale into the test-sample Breathbag

Kibion® Dynamic Base and Kibion® Dynamic Pro



Breathbag connection

